



Ospedale
di Cremona

Sistema Socio Sanitario



Regione
Lombardia
ASST Cremona

PORPHYRINS / CO-PORPHYRINS

URINE COLLECTION METHOD

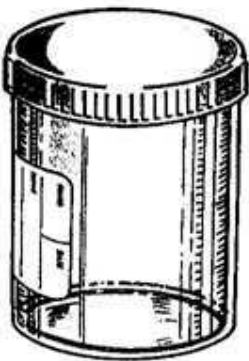
Prepare the collection container (fig. 1) **by wrapping it in aluminium foil.**

The container must be placed in a dark or semi-dark environment.

SAMPLE collection:

1. Record the patient's name and surname on the jar
2. The sample should be collected in a single urination, keeping the container away from light (at least 30 mL are required)
3. Deliver the tightly closed jar, covered with aluminium foil, to the chosen sample collection point during opening hours.

It is important to abide scrupulously by the instructions to ensure a successful test.

	<p>Prima della raccolta del campione, avvolgere il contenitore con carta stagnola.</p> <p>E' importante tenere al riparo dalla luce</p> <p>Il contenitore deve essere riempito almeno fino a 30 mL</p>
Contenitore	